# WIDEX **MOMENT™**

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### **GAMA DE PRODUSE WIDEX MOMENT™**

Modelul MBB2  
BTE  
(Retroauricular)

**WIDEX®**  
HIGH DEFINITION HEARING

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FCC ID: TTY-UFM

IC: 5676B-UFM

### Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

#### NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**NOTE:**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by Widex A/S could void the user's authority to operate the equipment.

**Declarația ISED / Déclaration d'ISED**

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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**ISED RADIATION EXPOSURE STATEMENT:**

This equipment complies with ISED RF radiation exposure limits set forth for an un-controlled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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**Nr. manual:**  
9 514 0577 033 01  
**Ediție:**  
2021-02

# WIDEX MOMENT™ BTE 312


The WIDEX BTE 312 is part of the WIDEX MOMENT™ family. Built around the WIDEX MOMENT™ platform to deliver better sound via TruAcoustics™ and the revolutionary Widex PureSound™ paradigm and all on a brand-new chip.

Fitting BTE 312 is easier than ever before, thanks to TruAcoustics™ giving a more powerful personalised sound. The WIDEX BTE 312 overcomes the challenges of fitting open/vented fits for mild-to-moderate hearing loss with the Widex PureSound™ paradigm.

- Multiple wireless connectivity via WidexLink technology and TONELINK App
- Compatible with the DEX assistive listening devices
- Uses a size 312 battery
- Protection class IP68
- Minimal to severe-to-profound hearing losses

## SUGGESTED FITTING RANGE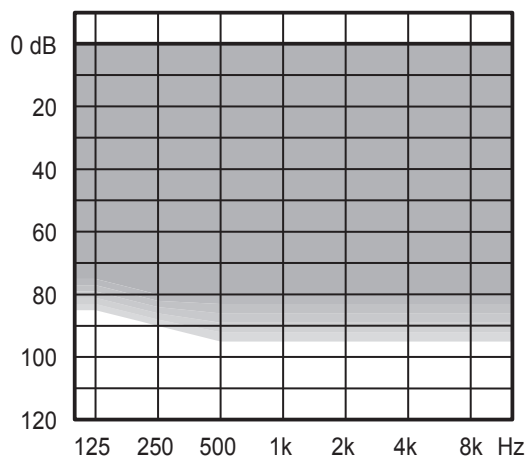

## STANDARD TECHNOLOGY

- WIDEX MOMENT™ platform with Personal Gain Integrator and ZeroDelay™ Technology
- Widex PureSound™ — new paradigm in sound processing
- TruAcoustics™ for improved sound and fitting

KEY FEATURES	440	330	220	110
Performance	.....	.....	....	..
Processing and fine-tuning channels	15	12	10	6
Widex PureSound™ with ZeroDelay™ Technology	•	•	•	•
TruAcoustics™ with Personal Gain Integrator	•	•	•	•

## CONNECTIVITY

WidexLink to DEX assistive listening devices*	440	330	220	110
	•	•	•	•

## APPS FOR iOS AND ANDROID


APP	440	330	220	110
TONELINK App	•	•	•	•
COM-DEX App	•	•	•	•

## FEATURES


FEATURE	440	330	220	110
Adaptation manager	•	•	•	•
Fluid Sound Analyser (sound classes)	11 (IE)	7 (IE)	4	3
Programs	5	4	3	3
Smartwind Manager	•			
High-frequency boost	•			
Speech Enhancer RT	RT/IE	IE		
Digital Pinna	•	•		
HD Locator	•	•	•	
TruSound Softener	•	•	•	
SoundSense Adapt	•	•	•	
Preference Control	•	•	•	•
Programmable Push Button**	•	•	•	•
Soft-level noise reduction	•	•	•	•
Noise Reduction	•	•	•	•
ZEN IE	•	•	•	•
Audibility Extender	•	•	•	•

\*Also includes DEX assistive listening devices: TV-DEX, COM-DEX, RC-DEX, FM+ DEX \*\*Programmable Preference Control, program shift or a combination of the two.


## MAXIMUM OUTPUT - EAR SIMULATOR

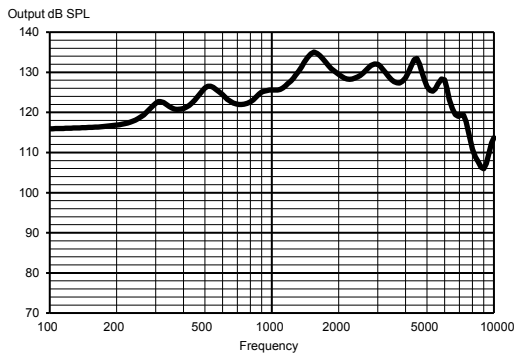
## MAXIMUM OUTPUT - 2CC COUPLER
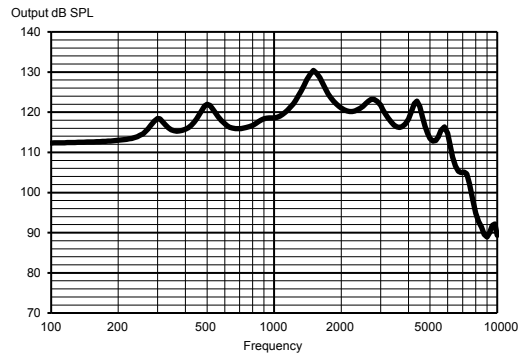
## OUTPUT - EAR SIMULATOR
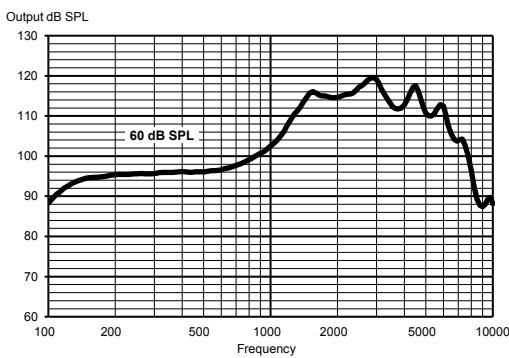
## OUTPUT - 2CC COUPLER
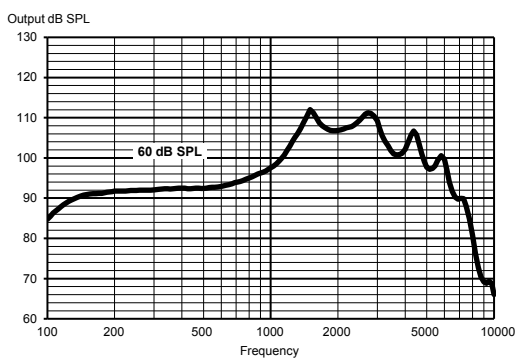
### Technical data:

Typical data obtained through standard pure tone measurements. Hearing aid set to Compass Reference Test Gain, unless stated otherwise. Measured using a BTE coupler and a standard hook. For further information, please contact Widex at [global.widex.com](http://global.widex.com).

		EAR SIMULATOR IEC 60118-0:1983 + A1:1994	2CC COUPLER IEC 60118-0:2015, ANSI S3.22-2014
OSPL90	1600 Hz	135 dB SPL	129 dB SPL
	Peak	135 dB SPL	130 dB SPL
	Average	127 dB SPL	123 dB SPL
Acoustic output (Input 60 dB SPL)	1600 Hz	116 dB SPL	110 dB SPL
	Peak	120 dB SPL	112 dB SPL
	Average	104 dB SPL	106 dB SPL
Full-on gain (Input 50 dB SPL, Compass Full-on gain)	1600 Hz	63 dB	56 dB
	Peak	64 dB	59 dB
	Average	58 dB	54 dB
Acoustic frequency range		100 Hz - 8450 Hz	100 Hz - 7750 Hz
Harmonic distortion (typical)	500 Hz	<3%	<3%
	800 Hz	<2%	<2%
	1600 Hz	<2%	<2%
Equivalent input noise		18 dB SPL	19 dB SPL
Battery drain (standby)		1.06 mA	1.06 mA
Battery drain*		1.11 mA	1.20 mA
Battery life (Type 312 Zn-Air, 170 mAh)*		155 h	140 h
Mobile phone immunity (IEC 60118-13:2016, ANSI C63.19:2011)		IRIL: -35/-31/-9 dB SPL	U-rating: M4

\*Battery life in real-life situations depends among other things on the hearing aid features used, streaming time, and the quality of the battery used.

Do not modify this equipment without authorization of the manufacturer. Spare parts and instructions for correct repair can be acquired from Widex.

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